



## Sample Collection Tube

### Instructions for Use (IFU)


#### Intended Use

AccuCollect™ Sample Collection Tube is designed for collection, transportation and short-term storage of clinical specimen.

#### Product Description

AccuCollect™ is a sample collection tube containing 3ml of transport media.

The transport media will lyse the organism and inactivate the cells, bacteria and virus. The stored samples can be used in subsequent molecular experiments such as DNA/RNA extraction, PCR testing or sequencing.

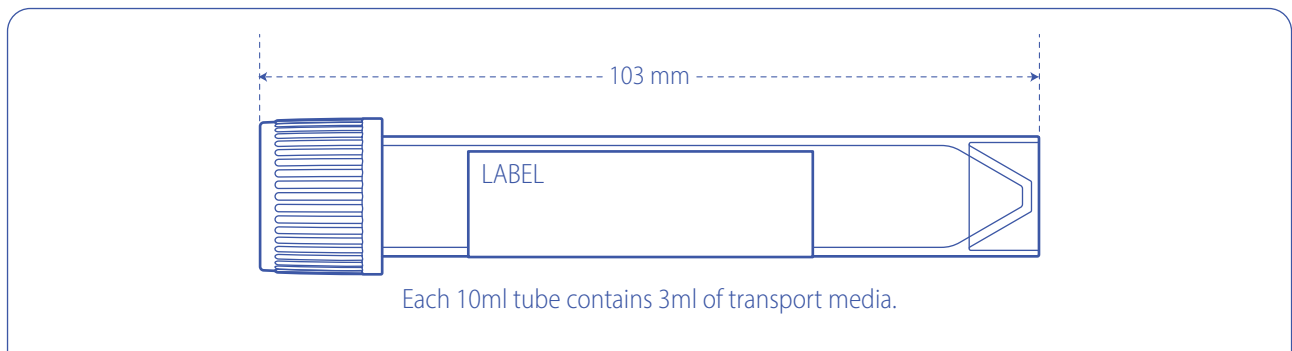

Description	AccuCollect™ Sample Collection Tube (Box of 50)
Tube Material	Polypropylene
Transport Media (Preservatives)	Hanks Balanced Salt Solution (HBSS), Heat-inactivated fetal bovine serum (FBS), antibiotic, antifungal
Product Code	102001-01
Additional Content	1 biohazard bag / tube




# AccuCollect™

## Sample Collection Tube


### Directions for Use

AccuCollect™ can be used with AccuSwab™, and should only be used by trained personnel following the descriptive diagram below:

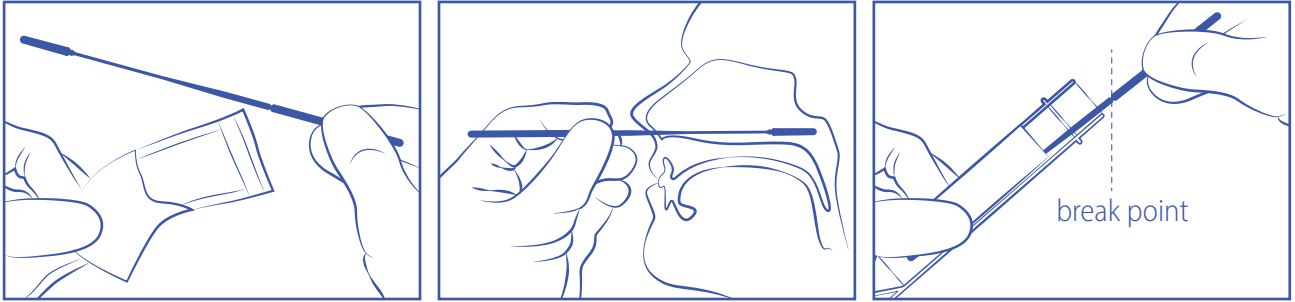

\* for use on adults only

1. Peel open sterile pouch to reveal AccuSwab™. Remove AccuSwab™ from pouch. Always hold at the swab handle with clean surgical gloves.
2. To minimize potential contamination, do not use AccuSwab™ for other human orifices except the patient's nose. Do not touch the swab tip.
3. Open AccuCollect™ and immerse the swab head in the preservatives, then break the swab handle at the break point to allow the swab to fit in the tube. Discard the swab handle.

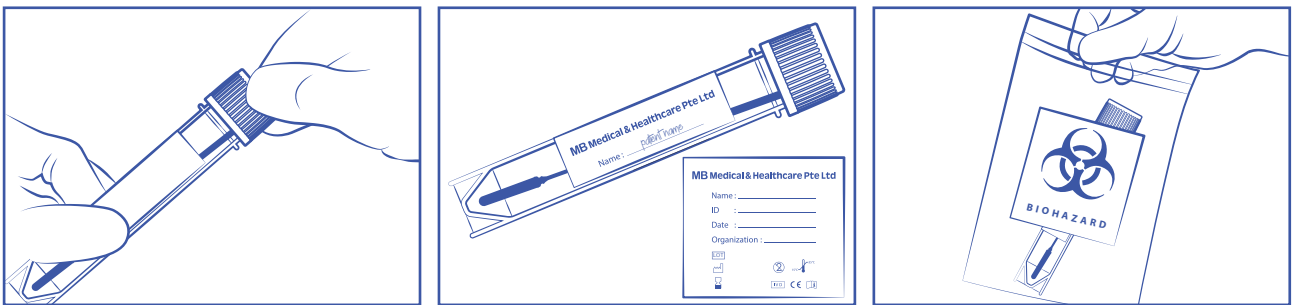

4. Tighten the cap on the tube.
5. Label the tube with sample information.
6. Place the sampling tube in a biohazard bag and seal it for transportation.





## Sample Collection Tube

### Storage

Before use, AccuCollect™ can be stored at 15–35°C for 24 months.

After collection, samples to be tested within 24 hours can be stored at 4°C. For longer storage, please store at -20°C and below.

### Symbol and Mark Key



Batch No.



Single Use



Date of Manufacture



Do not use if package is open or damaged



Expiry Date



Storage Temperature (15–35°C)

Distributed by:

**MB Medical & Healthcare Pte Ltd**

Invent Block Level 1  
26 Ang Mo Kio Industrial Park 2  
Singapore 569507

TEL: +65 6891 9988  
WEBSITE: [mbmedicalhealthcare.com](http://mbmedicalhealthcare.com)  
EMAIL: [info@mbmedicalhealthcare.com](mailto:info@mbmedicalhealthcare.com)



Our Innovation, Your Health