Sterile Swab Applicator

Instructions for Use (IFU)

Intended Use


AccuSwab® Sterile Swab Applicator is a nasopharyngeal swab applicator used for clinical sample collection. It is suitable for use in contact with the human body, in the procedure to diagnose infections. It should only be used as directed by trained personnel.

Product Description

AccuSwab¹ is injection molded with a bio compatible plastic that has excellent hydrophilic properties. It is designed with a unique fin feature (efficacy test proven) on the swab tip for optimal collection and elution of sample.

AccuSwab¹ 101001-01 comes in a breakpoint of 88mm to cater to most VTM tubes and has a smooth neck design.

AccuSwab¹ 101001-02 and 101001-04 (blue) comes in a breakpoint of 88mm to cater to most VTM tubes and has a tiered neck design as a mid-turbinate indicator, 50mm from the swab tip head.

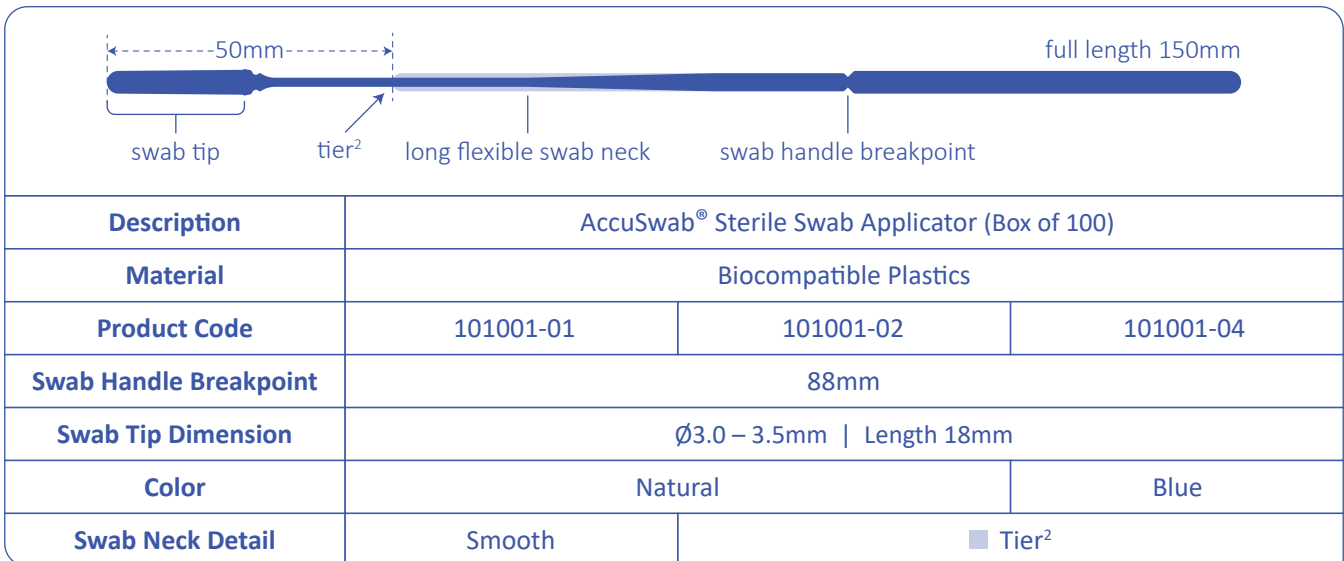¹ AccuSwab refers to AccuSwab® Sterile Swab Applicator

² Mid-turbinate indication

Sterile Swab Applicator

Precautions

1. Only to be used by trained personnel.
2. Sterile gloves and protective clothing and eyewear should be worn when collecting and handling microbiological samples with AccuSwab¹. Observe biosafety recommendations when handling or analyzing patient samples.
3. Reliable sample collection depends on many factors including collection and handling technique, sampling conditions and timing. Best results are achieved when swab samples are processed shortly after the time of collection.
4. AccuSwab¹ 101001-01, 101001-02 and 101001-04 are designed with an 88mm swab handle breakpoint for intentional breakage of the swab to fit into a 100mm transport tube. Do not use excessive force, pressure or bending when swabbing as this may result in accidental breakage.
5. Use with care when inserting into the nasopharyngeal cavity, as swabbing may cause mild trauma. Gently insert only as far as possible, and avoid forcing against resistance.

Warning

1. For single use only, do not re-use as it may cause a risk of infection and/or inaccurate results.
2. Do not re-pack. Do not re-sterilize.
3. Not suitable for any other application other than intended use.
4. The use of this product in conjunction with a rapid diagnostic test or instrument should be validated by the user.
5. AccuSwab¹ should not be used if peel pouch is damaged or if expiration date printed on label has passed.
6. AccuSwab¹ should not be used if it is visibly damaged (i.e., if the swab tip or swab neck is broken).
7. Do not use excessive force, pressure or bending when collecting samples from patients as this may result in accidental breakage of the swab.
8. Directions for use must be followed carefully. The manufacturer cannot be held responsible for any unauthorized or unqualified use of the product.
9. It must be assumed that all swab samples collected contain infectious microorganism and must be handled with appropriate precautions. Appropriate protective equipment should be worn. Laboratory and biosafety guidelines should be followed when handling a clinical swab sample.

Distributed by:

MB Medical & Healthcare Pte Ltd

Invent Block Level 1
26 Ang Mo Kio Industrial Park 2
Singapore 569507

TEL: +65 6891 9988
WEBSITE: mbmedicalhealthcare.com
EMAIL: info@mbmedicalhealthcare.com


Our Innovation, Your Health

Sterile Swab Applicator

Directions for Use

AccuSwab¹ should only be used by trained personnel following the descriptive diagram below: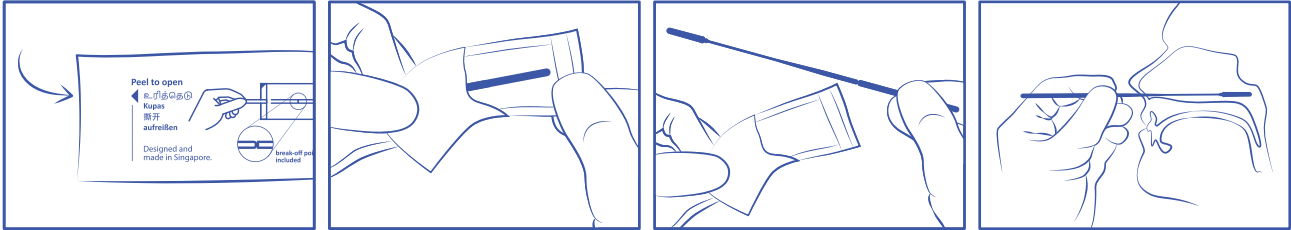

* for use on adults only

1. Check if pouch is damaged as any damage would affect the sterility of the swab.
2. If it's in good condition, proceed to peel open sterile pouch at the designated area to reveal AccuSwab¹.
3. Remove AccuSwab¹ from pouch. Always hold at the swab handle with clean surgical gloves. Do not touch the swab tip.
4. AccuSwab¹ should be processed immediately according to internal laboratory instructions. Alternatively, AccuSwab¹ may be placed in a sterile tube marked with patient information and transferred to laboratory for microbiological analysis.
5. Disposal of tubes and AccuSwab¹ must be done according to laboratory regulations for biohazard waste.

Storage

Store AccuSwab¹ in original packaging at room temperature. If the product is stored at a different temperature, it may not maintain the stated specifications.

Symbol and Mark Key

	Batch No.		Single Use
	Date of Manufacture		Do not use if package is open or damaged
	Expiry Date		Method of Sterilization (Ethylene Oxide)
	CE Mark		Medical Device
0123	Notified Body (NB) No.		Consult Instructions for Use

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